


## FINAL PLACEMENT OF IMPLANT



**1** Remove the tray from the implant box, and peel back the tray cover, exposing the implant container.



**2** Remove the implant container from the tray, and then remove its cap.

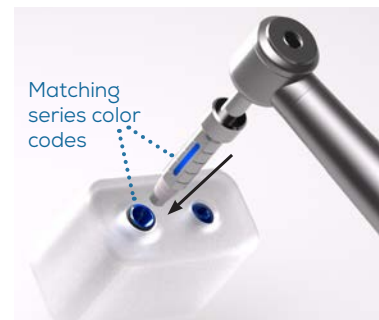


**3** Firmly press the implant driver (handpiece or ratchet) into the implant until it clicks. Then pull the implant out of the container. Be sure to use the driver that matches the series of the implant.



**4** Carefully rotate the implant into the osteotomy. The depth marks on the driver help you to gauge when you have reached the depth determined by the surgical plan. Do not exceed the maximum torque of 60 Ncm.



**5** If installing a cover screw, press an .050 Hex Driver from the Surgical Kit into the supplied cover screw, then lift it out of the container.

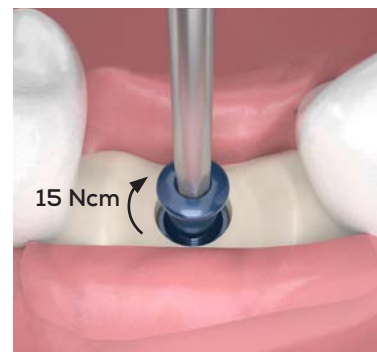

**6** Drive the cover screw or healing cap into the implant to a maximum torque of 15 Ncm or finger tighten. Close and suture if needed.



Alternatively, an .050 Hex Driver can be used to install a healing cap.

